



M Clayton
26/09/2017
V1
010007

PREPARED
DATE
VER NO.
PRODUCT CODE

Instruction Manual

For

IRiS Light Wand



experia
multisensory for everyone

Experia
Acorn Phase 3, High Street
Barnsley, South Yorkshire
S72 7BD, United Kingdom

T 0800 612 6077
Einfo@experia-innovations.co.uk
Wwww.experia-innovations.co.uk

Instruction Manual

Congratulations on the purchase of your IRiS Light Wand!

The IRiS Light Wand has been designed specifically for people with special needs and should, if used in accordance with these guidelines, offer years of switching learning and fun!

The IRiS Light Wand offers endless opportunities to develop numerous skills including colour recognition, cause and effect, hand to eye co-ordination and gross and fine motor skills and many more.

The IRiS Light Wand is an IRiS Talker and can control any number of IRiS Listeners.

Please ensure you read this Instruction Manual in full before operating the unit.

Heed all warnings on the product and within this manual. If upon receipt of this product, the user suspects that the product is defective, and/or requires service or repair due to malfunction or damage to the product, please contact the manufacturer immediately. Please be aware that improper installation and/or use or misuse of this product may result in injury, death, or adverse reactions to its user. This product is intended to be installed and/or use by a trained professional. Failure to comply with these precautions or with specific warnings elsewhere in this manual violates safety standards of design, manufacture, and intended uses of the product.

Features

- IRiS Wirefree Compatible
- Sensitive Joystick Control
- Robust Design
- Jack input for external switch (3.5mm)
- Durable
- Easy to use
- 3 Year Manufacturers Guarantee (Return to base.)

Description

The IRiS Light Wand works with all IRiS Listeners but is specifically designed to work with the IRiS Scanner or IRiS Musical Touch Wall.

Every time you press the switch on the top, any product paired to the unit will change colour.

If you have paired the IRiS Light Wand to the IRiS Scanner, every time you move the joystick, the light from the IRiS Scanner will move.

Instruction Manual

Getting Started

1. Decide which IRiS Listeners you want to control and pair the unit to them – see ‘Pairing to IRiS Listeners’ section.
2. Now it’s ready to use: simply press the switch
3. Move the joystick (only works with IRiS Scanner & IRiS Musical Touchwall).

Using your own switches

Connect your own switches by plugging them into the socket on the side of the unit.

Pairing to IRiS Listeners

This procedure is split into 2 sections, one for those who do not have an IRiS Master Pairer and for those who do.

Without an IRiS Master Pairer

Every IRiS Listener has a button on it called a pairing button. To get the IRiS Light Wand to work with a product, first find the pairing button on the product and press it. You then need to press the switch on the top of the IRiS Light Wand within 5 seconds. The IRiS Listener will change colour every time you press this switch if correctly paired. The unit will now work with the IRiS Light Wand.

If you have paired the IRiS Light Wand to the IRiS Scanner or IRiS Musical Touch Wall, movement of the joystick will control the movement of the light on the IRiS Listener.

With an IRiS Master Pairer

To get the IRiS Light Wand to work with a product, first decide which unit you want to control and find its pairing button on the IRiS Master Pairer and press it. You then need to press the switch on the top of the IRiS Light Wand within 5 seconds. The IRiS Listener will change colour every time you press this switch if correctly paired. The unit will now work with the IRiS Light Wand. If you have paired the IRiS Light Wand to the IRiS Scanner or IRiS Musical Touch Wall, movement of the joystick will control the movement of the light on the IRiS Listener.

Instruction Manual

Cleaning and Maintenance

The unit should be cleaned with a damp soft cloth. A non-abrasive cleaning agent may be used for stubborn stains. Ensure no liquid enters the unit.

The unit has been designed for use by people with special needs however this does NOT mean it is unbreakable. You must ensure the unit is not dropped, thrown, stood upon, immersed in water, etc.

Troubleshooting

Q. The unit doesn't work.

A1. Is it switched on? If so, see A2

A2. Has the battery been changed? If so, please contact us.

Q. The unit won't control any product.

A1. Is it switched on? If so see A3

A2. Is the unit paired to an IRiS Listener? If so, see A4

A3. Try pairing the unit to another IRiS Listener: does this work? If not, please contact us.

Instruction Manual

Contact Details

For USA please email office@experia-usa.com

For Ireland please email support@experia-ireland.ie

For UK and rest of the world email support@experia-innovations.co.uk

Telephone: +44 (0) 845 644 0977.

Please note: email is our support teams preferred communication and this takes precedence over telephone calls.

QUESTIONS & IDEAS?

If you have an idea on how to improve this product or want to see your product idea realised, please email: ideas@experia-innovations.co.uk.

All ideas are presented to our OT, technical and finance department and if all procedures are passed we will contact you to discuss your idea further.

We can't promise to develop every product but if it's a great idea, we'll try!

Manufactured proudly in the UK by:

Experia, Acorn Phase 3, High Street, Grimethorpe, S72 7BD.



© Copyright 2017 Experia All rights reserved. Experia is a registered trademark