



M Clayton	PREPARED
08/07/2020	DATE
V1	VER NO.
01 00 62 2	PRODUCT CODE

# Instruction Manual

For  
IRiS Colour  
Selector  
Deluxe  
Version2,  
wireless  
charging

experia  
multisensory for everyone

Experia  
Acorn Phase 3, High Street  
Barnsley, South Yorkshire  
S72 7BD, United Kingdom

T 0800 612 6077  
Einfo@experia-innovations.co.uk  
Wwww.experia-innovations.co.uk

# Instruction Manual

**Congratulations** on the purchase of your IRiS Colour Selector Deluxe!

The IRiS Colour Selector Deluxe has been designed specifically for people with special needs and should, if used in accordance with these guidelines, offer years of switching learning and fun!

The IRiS Colour Selector Deluxe offers endless opportunities to develop numerous skills including colour recognition, cause and effect, hand to eye co-ordination and gross and fine motor skills and many more.

The IRiS Colour Selector Deluxe is an IRiS Talker and can control any number of IRiS Listeners.

**Please ensure you read this Instruction Manual in full before operating the unit.**

Heed all warnings on the product and within this manual. If upon receipt of this product, the user suspects that the product is defective, and/or requires service or repair due to malfunction or damage to the product, please contact the manufacturer immediately. Please be aware that improper installation and/or use or misuse of this product may result in injury, death, or adverse reactions to its user. This product is intended to be installed and/or use by a trained professional. Failure to comply with these precautions or with specific warnings elsewhere in this manual violates safety standards of design, manufacture, and intended uses of the product.

## Features

- IRiS wirefree compatible
- 8 Switching Modes for lifeskills development.
- LCD Display
- Easy to clean membrane switch panel
- Integral 3.5mm switch input.
- Wireless Charging.
- No batteries to swap! It is wireless charging!
- Durable.
- Ergonomic Design.
- Easy to use
- 3 Year Manufacturers Guarantee (Return to base)

# Instruction Manual

## Pack Contents

1	x	IRiS Colour Selector Deluxe
1	x	Charging Cradle
1	x	USB power adapter
1	x	Instruction Manual

## Getting Started

1. Before first use, it is recommended to fully charge the IRiS Colour Selector Deluxe.
2. Place the charging cradle on a flat surface, with the black pad uppermost.
3. Connect the USB cable, to the USB power adapter, and connect to a suitable mains outlet.
4. Place the IRiS Colour Selector Deluxe, onto the cradle, so the four locating pins on the cradle, fit into the four holes on the underside of the IRiS Colour Selector Deluxe.
5. Turn on the power to the USB power adapter.
6. There is a charging light on the front of the IRiS Colour Selector Deluxe. This will illuminate, to indicate the unit is charging. When the charging light goes out, the IRiS Colour Selector Deluxe is fully charged.
7. NOTE – If the unit has not been used for a while, the charging LED may not illuminate immediately, please leave the unit on charge to get some power into the internal battery.
8. Once the unit is fully charged, simply press any of the coloured buttons to control the product it is paired to.
9. To pair the IRiS Colour Selector Deluxe to a product, please see the 'Pairing to IRiS Listeners' section in this manual.

## Using your own switches

To use your own switches, simply plug them into the colour coded sockets on the front of the IRiS Colour Selector Deluxe.

## Switching Modes

The IRiS Colour Selector Deluxe features 8 switch modes, each designed to aid the development of basic life skills.

To change the switch mode, simply press the button marked mode, located on the front of the unit. Each time this button is pressed the mode moves onto the next. The LCD display will show which mode the IRiS Colour Selector Deluxe is currently in.

# Instruction Manual

**Locked** – Press any colour on the front of the IRiS Colour Selector Deluxe, and the Listener paired to the unit will change to that colour and stays on that colour until a different colour is pressed. The listener then changes to the new colour.

**Toggle** - Press any colour on the front of the IRiS Colour Selector Deluxe, and the Listener paired to the unit changes to that colour. Press any colour again and the light extinguishes.

**Momentary** – Press and hold, any colour on the front of the IRiS Colour Selector Deluxe, and the Listener paired to the unit will change to that colour. The moment you release that colour, the lights extinguish.

**Racing** - This mode operates like a quiz buzzer. The first coloured button pressed illuminates the Listener paired to the unit and stops any other switch action. After a selected time, the colour extinguishes and it is ready to start again.

Times available are: 1, 2, 3, 4, 5, 10, 15, 20, 25, 30, 40, 50, 60 seconds, using the time button.

**Timed** – Press any colour on the front of the IRiS Colour Selector Deluxe, and the Listener paired to the unit will change to that colour for a pre-set time. The time is fully adjustable by pressing the 'Time' button on the front of the IRiS IRiS Colour Selector Deluxe.

Times available are: 1, 2, 3, 4, 5, 10, 15, 20, 25, 30, 40, 50, 60 seconds.

**Scrolling Stop** - The Listeners paired to the unit start changing colour. Press any colour button on the front of the IRiS Colour Selector Deluxe to stop on the colour illuminated at that point. The Listeners stays on that colour for a present time. The time is fully adjustable by pressing the 'Time' button on the unit. Each time you press the 'Time' switch the time increases.

Times available are: 1, 2, 3, 4, 5, 10, 15, 20, 25, 30, 40, 50, 60 seconds.

**Scrolling Next** – When any coloured button is pressed the Listeners paired to the unit will scroll to the next colour.

**Passive** – The Listeners paired to the unit start slowly changing colour. Ideal for a relaxation session.

# Instruction Manual

## Pairing to IRiS Listeners

This procedure is split into 2 sections, one for those who do not have an IRiS Master Pairer and for those who do.

### *Without a IRiS Master Pairer*

Every IRiS Listener has a button on it called a pairing button. To get the IRiS Colour Selector Deluxe to work with a product, first find the pairing button on the product and press it. You then need to press a colour button on the top of the IRiS Colour Selector Deluxe within 5 seconds. The IRiS Listener will now react to the IRiS Colour Selector Deluxe each time a colour button is pressed.

### *With a IRiS Master Pairer*

To get the IRiS Colour Selector Deluxe to work with a product, first decide which IRiS Listener you want to control and find its pairing button on the IRiS Master Pairer and press it. You then need to press a colour button on the top of the IRiS Colour Selector Deluxe within 5 seconds. The IRiS Listener will now react to the IRiS Colour Selector Deluxe each time a colour button is pressed.

## Charging the IRiS Colour Selector Deluxe

1. Place the charging cradle on a flat surface, with the black pad uppermost.
2. Connect the USB cable, to the USB power adapter, and connect to a suitable mains outlet.
3. Place the IRiS Colour Selector Deluxe, onto the cradle, so the four locating pins on the cradle, fit into the four holes on the underside of the IRiS Colour Selector Deluxe.
4. Turn on the power to the USB power adapter.
5. There is a charging light on the front of the IRiS Colour Selector Deluxe. This will illuminate, to indicate the unit is charging. When the charging light goes out, the IRiS Colour Selector Deluxe is fully charged.
6. NOTE – If the unit has not been used for a while, the charging LED may not illuminate immediately, please leave the unit on charge to get some power into the internal battery.

# Instruction Manual

## Cleaning and Maintenance

The unit should be cleaned with a damp soft cloth. A non-abrasive cleaning agent may be used for stubborn stains. Ensure no liquid enters the unit.

The unit has been designed for use by people with special needs, however, this does NOT mean it is unbreakable. You must ensure the unit is not dropped, thrown, stood upon, immersed in water, etc.

## Troubleshooting

Q. The IRiS Colour Selector Deluxe doesn't work.

1. Does the LCD display illuminate when a button is pressed? If not,
2. Has the unit been charged? if so,
3. Use a paperclip to reset the unit. The reset hole is on the bottom.
4. Please contact us.

Q. The IRiS Colour Selector Deluxe will not control an IRiS Listener.

1. When you press a button on the unit, does the LCD display show text
2. Is the unit paired to an IRiS Listener? If so
3. Try pairing the unit to another IRiS Listener, does this work? If not, please contact us.

# Instruction Manual

## Contact Details

For USA please email [office@experia-usa.com](mailto:office@experia-usa.com)

For UK and rest of the world email [support@experia.co.uk](mailto:support@experia.co.uk)

Telephone: +44 (0) 845 644 0977.

Please note: email is our support team's preferred communication and this takes precedence over telephone calls.

## QUESTIONS & IDEAS?

If you have an idea on how to improve this product or want to see your product idea realised, please email: [ideas@experia.co.uk](mailto:ideas@experia.co.uk).

All ideas are presented to our OT, technical and finance department and if all procedures are passed, we will contact you to discuss your idea further.

We can't promise to develop every product but if it's a great idea, we'll try!

## Manufactured proudly in the UK by:

Experia, Acorn Phase 3, High Street, Grimethorpe, S72 7BD.



© Copyright 2017 Experia All rights reserved. Experia is a registered trademark