

Monkey Climber

43 00 60

Product Manual



experia
multisensory for everyone

Experia
Acorn Phase 3, High Street
Barnsley, South Yorkshire
S72 7BD, United Kingdom

T 0800 612 6077
E info@experia-innovations.co.uk
W www.experia-innovations.co.uk

Congratulations on the purchase of your Monkey Climber!

The Monkey Climber is a wonderful addition to our product range.

It can be used as part of an obstacle course to address gross motor coordination as well as provide proprioceptive input.

Please ensure you read and follow these Instructions carefully before installing or operating the unit. Please keep this manual for future reference.

Heed all warnings on the product and within this manual. If upon receipt of this product, the user suspects that the product is defective, and/or requires service or repair due to malfunction or damage to the product, please contact the manufacturer immediately. Please be aware that improper installation and/or misuse of this product may result in injury, death, or adverse reactions to its user. This product is intended to be installed and/or used by a trained professional. Failure to comply with these precautions or with specific warnings elsewhere in this manual violates safety standards of design, manufacture, and intended use of the product.

We recommend the use of padding around this unit - available separately.

Max weight for any Monkey Climber unit is: 200kg.

Tools required = 13mm Spanner (not supplied)

Features

- Stylish design.
- Made from strong Russian Birch timber.
- Modular and easy to set up.
- Self-assembly required - should take 15 minutes with 2 people.
- It can be used with various associated units to help develop numerous skills.
- Positive tactile experience.
- Provides proprioceptive input.

Pack contents

1x Ladder Top
2x Ladder Sides
4x Feet
32x Bolts
32x Washers
32x Safety Caps

Unpacking your Monkey Climber






Check packaging for signs of damage before opening.
Do not use sharp objects during unpacking.
Check the parts received against the Pack Contents.
Read the product manual before use.

43_00_60

MONKEY CLIMBER

Instructions:

1. Fix feet to outsides of both ladder legs using 16 x bolts, 16 x washers. Tighten securely.
2. Fix ladder top to outside faces of ladder legs with 16 x bolts and washers. Tighten securely.
3. Push 32 x safety caps over each bolt head.
4. Ensure all bolt heads are safely covered prior to use

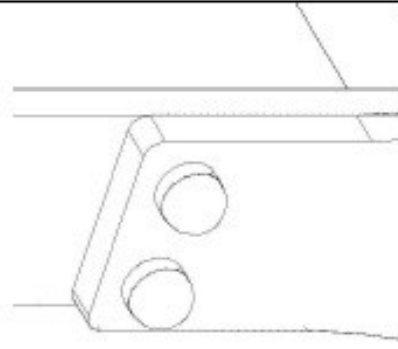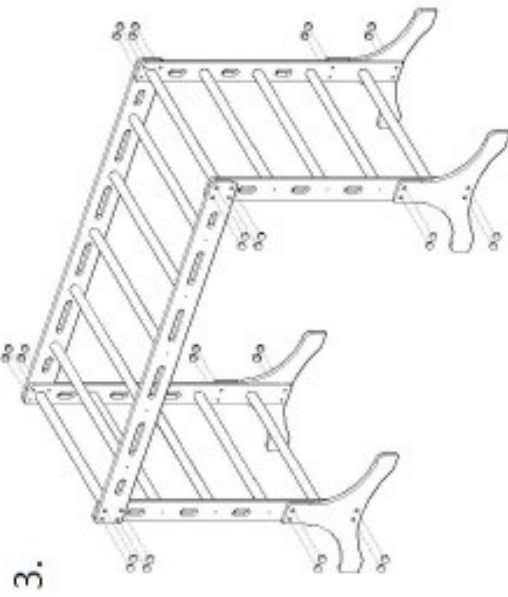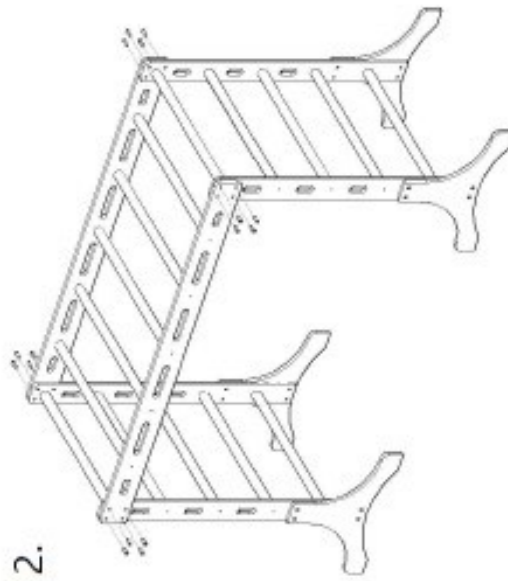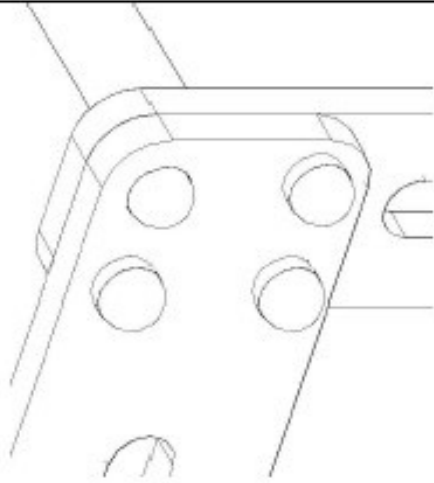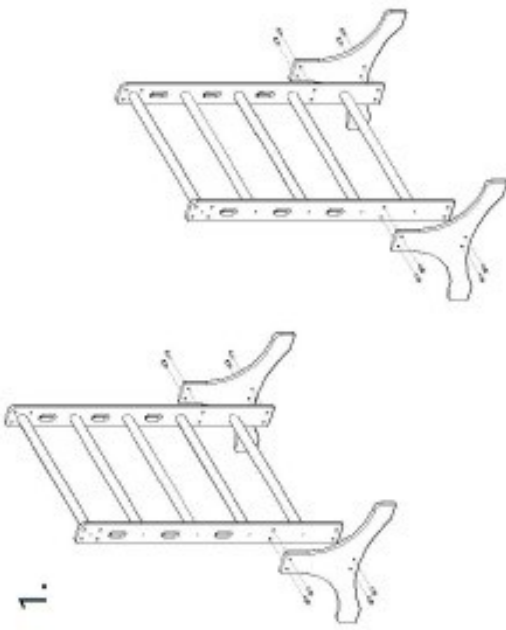

- Contents**
- 2 x Ladder Sides / 1 x Ladder Top / 4 x Feet
 - 32 x Bolts, 32 x Washers
 - 32 x Safety Caps

Assembly Instructions

Tools required for assembly; 13mm Spanner (not supplied)

Fixings supplied

32x M8 x35mm Hex Bolts.

32x M8 Form A Washers Zinc plated.

32x M8 Black Safety nut cap.

Assembly

There are four holes on each end of the Ladder Sides with threaded inserts.

The hole spacing on one end match the holes on the feet.

The holes at the other end are for the Ladder Top.

Put a washer on each bolt.

Place the feet on the outside of the Ladder Sides

Match the holes in the feet with the Ladder Sides.

Put the bolt through the feet into the Ladder Sides.

Start all four bolts, then tighten them securely using a 13mm spanner.

Attach the foot on the opposite side in the same way.

Assemble the other Ladder Side in the same way.

Attach one end of the Ladder Top to the end of a Ladder Side putting the bolt / washer through into the threaded insert.

Start all bolts, then tighten them securely using a 13mm spanner.

Tighten all bolts securely.

Push the safety nut caps over each exposed bolt head.

Cleaning and Maintenance

It is advisable to check for splinters before every planned use of the Monkey Climber.
On a weekly basis check for loose bolts and tighten any found.

The unit should be cleaned with a damp soft cloth. A non abrasive cleaning agent may be used for stubborn stains.
No sharp objects should come into contact with the item at any time.

Safety Precautions

Under no condition should this equipment be used without proper supervision.

Appropriate spotting / safety techniques should be used at all times.

The safety of the Monkey Climber users is dependent upon the appropriate choice of activity to match or challenge their physical abilities.

The layout of your activity area, proper installation and maintenance of your equipment is essential.

Experia Sensory Integration equipment is not intended for recreational or outdoor use.

Technical Specifications

Dimensions: 1800L x 830W x1320mmH.

Max load weight for any Monkey Climber unit is: 200kg.

Trouble Shooting

If you suspect anything is wrong with your sensory integration equipment supplied by Experia and you have constructed it correctly following these instructions then please contact our office using the detail supplied at the end of this document.

Guarantee

All products manufactured by ourselves are backed with a three year guarantee.

All other items carry a manufacturers 1 year guarantee.

All guarantees are collect and return.

If your product needs to be returned to our factory we will arrange collection of the item and return it to you when repaired, all we ask is you pack the product securely.

If in the unlikely event that a product purchased from us develops a fault please contact us on

Freephone 0800 612 6077 or [email us](#) and our technical team will assist

Experia Registered Office:

**Acorn Phase 3
High Street
Grimethorpe
S72 7BD**

If you have any questions or need information about any of our products and services please call us on

Freephone 0800 612 6077

If you would like a FREE Sensory Brochure you can request one by telephone or visiting our website.

For more information on Experia Free 3D Room Designs or to schedule an onsite visit from our sales team then please request a visit by telephone, email or by using the form on our website.

Support Contact Details

For USA please email support@experia-usa.com

For Canada please email support@experia-canada.com

For UK and rest of the world email support@experia-innovations.co.uk

Telephone: 0044 (0) 800 612 6077.

Please note: email is our support teams preferred communication and this takes precedence over telephone calls.

Manufactured in the UK by:

Experia, Acorn Phase III, High Street, Grimethorpe, Barnsley, South Yorkshire, S72 7BD



© Copyright 2015 Experia All rights reserved Experia. Experia is a registered trademark