



3+ years



**WARNING:**

CHOKING HAZARD – Small Parts  
Not suitable for children under 3 years.



# IRiS

## Single Switch

### Instruction Manual

**Congratulations** on the purchase of your IRiS Single Switch!

The IRiS Single Switch has been designed specifically for people with special needs and should, if used in accordance with these guidelines, offer years of switching learning and fun!

The IRiS Single Switch offers endless opportunities to develop numerous skills including colour recognition, cause and effect, hand to eye co-ordination, gross and fine motor skills and many more.

The IRiS Single Switch is an IRiS Talker and can control any number of IRiS Listeners.

Put simply, it is operated by pressing the switch on the top of the unit.

**Please ensure you read this Instruction Manual in full before operating the unit.**

#### Features

- IRiS Wirefree Compatible
- Brightly Coloured Switch
- 6.5mm Switch Input Socket
- 6.5mm to 3.5mm Jack Adaptor
- Rugged Switch
- 4 x AA Batteries
- Easy to use
- 2 Year Manufacturers Guarantee (Collect and Return)

#### Getting Started

1. Press the switch on the IRiS Single Switch – the unit will control the products it is paired to.
2. To pair the IRiS Single Switch to a product please see the 'Pairing to IRiS Listeners' section in this manual.
3. Whenever the switch is pressed any IRiS Listener paired to the unit will change colour.

#### Using your own switches

Connect your own switch by plugging it into the socket on the front of the unit. A 6.5mm to 3.5mm jack adaptor is supplied with the unit to ensure you can connect your own switch.

## Pairing to IRiS Listeners

This procedure is split into 2 sections, one for those who do not have an IRiS Master Pairer and for those who do.

### *Without a IRiS Master Pairer*

Every IRiS Listener has a button on it called a pairing button. To get the IRiS Single Switch to work with a product, first decide which product you want to control then find the pairing button on that product and press it. You then need to press the switch on IRiS Single Switch within 5 seconds. The IRiS Listener will now change colour every time you press a switch if correctly paired. The unit will now work with the IRiS Single Switch.

### *With a IRiS Master Pairer*

To get the IRiS Single Switch to work with a product, first decide which unit you want to control and find its pairing button on the IRiS Master Pairer and press it. You then need to press the switch on IRiS Single Switch within 5 seconds. The IRiS Listener will now change colour every time you press a switch if correctly paired. The unit will now work with the IRiS Single Switch.

## Cleaning and Maintenance

The unit should be cleaned with a damp soft cloth. A non abrasive cleaning agent may be used for stubborn stains. Ensure no liquid enters the unit.

The unit has been designed for use by people with special needs however this does NOT mean it is unbreakable. You must ensure the unit is not dropped, thrown, stood upon, immersed in water, etc.

## Troubleshooting

Q. The unit doesn't work.

A1. When the switch is pressed on the unit does a blue LED illuminate? If not,

A2. Replace the batteries

A3. If the unit does not operate correctly after completing the above please contact us.

Q. The unit won't control any product.

A1. When the switch is pressed on the unit does a blue LED illuminate? If not,

A2. Replace the batteries

A3. Is the unit paired to an IRiS Listener? If so,

A4. Try pairing the unit to another IRiS Listener, does this work? If not, please contact us.

For support please email [support@experia-innovations.co.uk](mailto:support@experia-innovations.co.uk) or telephone 0845 644 0977. Please note: email is our support teams preferred communication and this takes precedence over telephone calls.



Copyright Experia 2008